Dräger

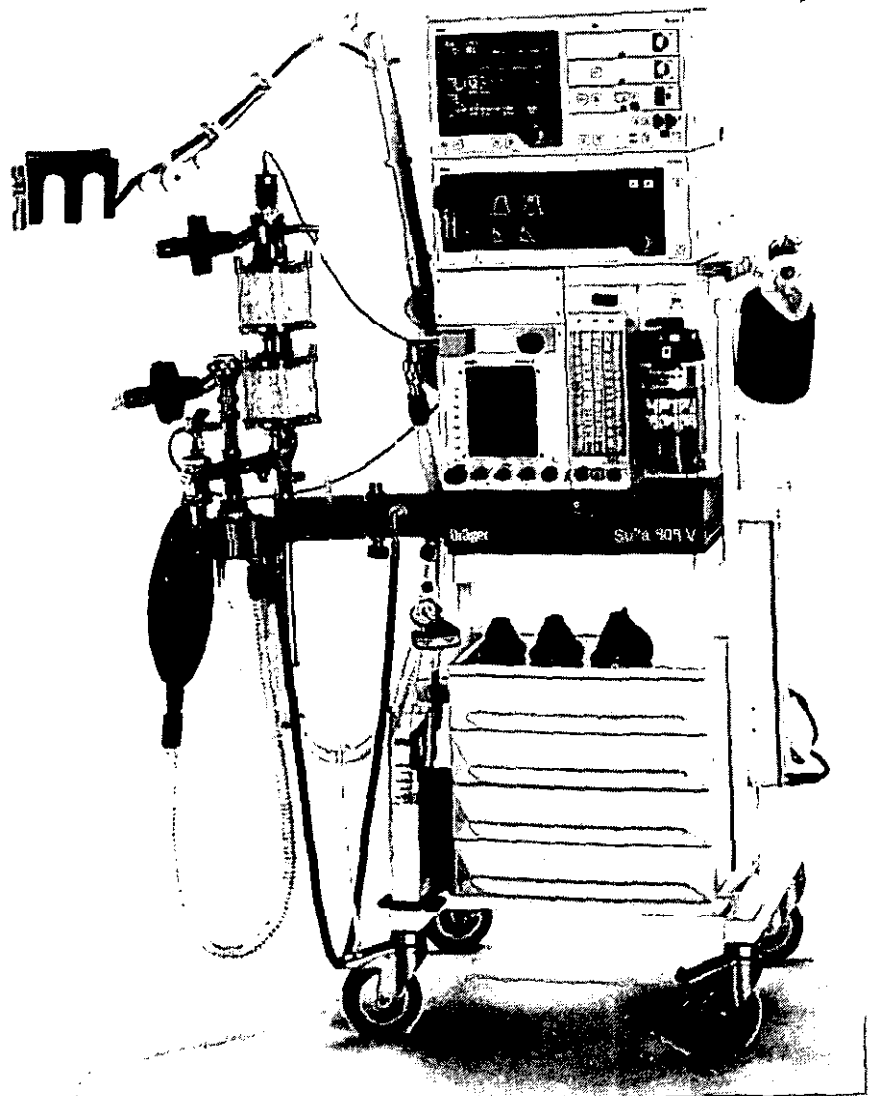
Sulla 909 V / 909 V-D

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Inhalation Anaesthetic Machines

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Instructions for Use

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	Page
For Your Safety and That of Your Patients.....	3
Intended Use.....	6
Machine Check.....	7
Operation.....	12
Operating modes.....	12
Controlling fresh-gas flow.....	13
O ₂ flush when necessary.....	13
Bronchial suction device.....	13
Low Flow anaesthesia.....	14
What to do during gas shortage.....	15
Shut-down.....	16
Care.....	17
Stripping down.....	17
Cleaning and disinfecting.....	17
Wipe disinfecting.....	18
Cleaning and disinfecting in machines.....	18
Sterilizing.....	18
Assembling Machine.....	19
Connecting to medical gas supply.....	19
Using and connecting O ₂ and N ₂ O cylinders.....	19
Connecting Circle System 9.....	20
Medical gas supply for Ventilog.....	20
Ventilation with Ventilog 3.....	21
Ventilation with Ventilog 2.....	21
Fitting Vapor 19.n.....	22
Fitting airway monitor.....	23
Connecting sensors to airway monitor.....	23
Fitting Oxydig.....	24
Fitting scavenging system.....	25
Attaching manual ventilation bag.....	25
Potential equalisation.....	25
Preparing brochial suction device.....	26
Moving Sulla 909 V-D.....	27
Checking Readiness for Operation.....	28
Checking medical gas connections.....	28
Checking safety devices.....	28
Checking switch – for Air supply.....	29
Checking gas type.....	30
Leak testing.....	31
O ₂ flush.....	31
Circle System 9.....	31
Ventilog.....	32
Bronchial suction device.....	32
Vapor 19.n.....	32
Scavenging system.....	32
Manual ventilation bag for emergency ventilation.....	33
PM 8050 or PM 8030 airway monitors.....	33
Maintenance Intervals.....	34
Appendix.....	35
Diagram of operating principles.....	35
Technical data.....	37
Mixing table.....	41
Explanation of abbreviations and terms used.....	42
Index.....	43

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For Your Safety and That of Your Patients

For correct and effective use of the apparatus and to avoid hazards it is essential to read the following recommendations and to act accordingly¹⁾:

Strictly follow the Instructions for Use

Any use of the apparatus requires full understanding and strict observation of these instructions. The apparatus is only to be used for purposes specified here.

Maintenance

The apparatus must be inspected²⁾ and serviced²⁾ regularly by trained service personnel at six monthly intervals (and a record kept). Repair²⁾ and general overhaul of the apparatus may only be carried out by trained service personnel. We recommend that a service contract be obtained with DrägerService and that all repairs also be carried out by them. Only authentic Dräger spare parts may be used for maintenance²⁾. Observe chapter "Maintenance Intervals".

Power connection

The apparatus is to be used only in rooms with mains power supply installations complying with national safety standards (such as in F. R. of Germany: VDE 0107).

The requirements laid down in IEC 601-1 "Safety of Medical Electrical Equipment" are applicable for electrically powered equipment.

1)

Insofar as reference is made to laws, regulations or standards, these are based on the legal system of the Federal Republic of Germany.

2)

Definitions according to DIN 31 051:

Inspection = examination of actual condition

Service = measures to maintain specified condition

Repair = measures to restore specified condition

Maintenance = inspection, service, repair

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12.92

Not for use in areas of explosion hazard

This apparatus is neither approved nor certified for use in areas where combustible or explosive gas mixtures are likely to occur.

Safe connection with other electrical equipment

Electrical connections to equipment which is not listed in these Instructions for Use should only be made following consultations with the respective manufacturers or an expert.

Liability for proper function or damage

The liability for the proper function of the apparatus is irrevocably transferred to the owner or operator to the extent that the apparatus is serviced or repaired by personnel not employed or authorized by DrägerService or if the apparatus is used in a manner not conforming to its intended use.

Drägerwerk Aktiengesellschaft cannot be held responsible for damage caused by non-compliance with the recommendations given above. The warranty and liability provisions of the terms of sale and delivery of Drägerwerk Aktiengesellschaft are likewise not modified by the recommendations given above.

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Safety procedures for users in Germany

According to DIN 13 252 and MedGV the following have to be monitored:

airway pressure,
expiratory minute volume,
inspiratory O₂ concentration,
anaesthetic agent concentration,
inspiratory and expiratory CO₂ concentration.

Undesirable changes in these parameters may for example be due to:

sudden changes in the patient's condition,
machine faults, such as leaks,
component failure,
failure of power or gas supplies, operating faults.

Suitable monitoring equipment is listed in the Design Licence under "Accessories required for monitoring". When monitoring equipment not on this list is used, the user must check that the equipment conforms to legal requirements and that it is suitable for monitoring the performance of the anaesthetic ventilator with reliability.

DIN 13 252 also requires that there is a manual ventilation system which is independent of the anaesthetic ventilator so that the patient can be adequately ventilated, with ambient air. If an anaesthetic ventilator is found to have a fault which means that its life-saving function can no longer be guaranteed, the patient must be ventilated with an independent manual ventilation device immediately.

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Intended Use

Inhalation anaesthetic machines, with a continuous flow of fresh gas, which deliver minimal flows below 1 L/min, for operating theatres, and induction and recovery rooms –

for use with O₂, N₂O and Air supplied by a medical gas pipeline system –

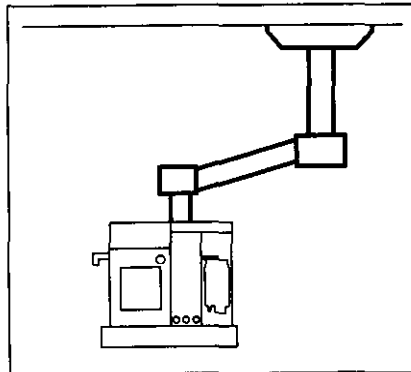
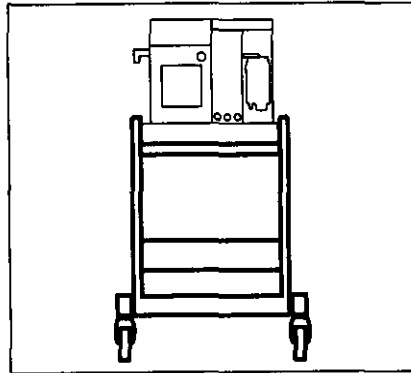
with additional connections for reserve cylinders of O₂ and N₂O – 3 or 11 litres –

with the pneumatic Ventilog® anaesthetic ventilator for adults, and for children with a body weight of no less than about 5 kg.

The machines conform to DIN 13 252 and fall within machine group 1 of the Medical Equipment Regulations.

Top illustration: **Sulla 909 V** with Ventilog® anaesthetic ventilator, on trolley with cupboard.

Bottom illustration: **Sulla 909 V-D** with Ventilog® anaesthetic ventilator, on ceiling pendant unit.

Beware – risk of explosion

Do not use with flammable anaesthetic agents such as ether, cyclopropane or similar.

Do not oil or grease cylinder valves on oxygen cylinders and pressure regulators for oxygen; do not touch with greasy fingers.

Do not store oxygen cylinders along with flammable substances.

Protect full steel cylinders from direct heat – do not store in direct sunlight – do not store close to radiators.

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Handwheels and knurled screws must only be tightened and loosened by hand.

Do not use force. Do not use any tools. Valve handwheels and knurled screws are precision parts which may be damaged if force is applied.

Leaking and stiff cylinder valves and stiff knurled screws must be repaired by trained service personnel.

Observe Instructions for Use for ancillary equipment carefully:

re-breathing system,
such as Circle System 9 –
anaesthetic vaporiser,
such as Vapor 19.n – anaesthetic
ventilators,
such as Ventilog 2 or Ventilog 3 –
airway monitor,
such as PM 8030 or PM 8050

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Machine Check

Sulla 909V
after care procedures.....page 17
after assembly.....page 19
after readiness for operation
has been checked.....page 28

Carry out machine check –
immediately before each use –
using checklist for machine:

Enter any changes and additions, as
well as the model description and
serial number, with a water-resistant
felt-tip pen –
Enter check results in pencil –
sign (top right), rub out entries before
next check.

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Dräger

Checklist for Inhalation anaesthetic machines with ORC Low Flow safety device and Circle System 9

Date _____

Model: _____
Serial No. _____

- Knowledge of relevant Instructions for Use is essential
- Delete whatever does not apply; make additions where necessary

Signature _____

Always check just before us

What	How	Desired	Actual
Anaesthetic gas			If in order tick
Cylinder supply	Open valves	Pressure O ₂ > 50 bar, N ₂ O > 30 bar	_____
Pipeline system	Insert probes		_____
Scavenging system	Insert probes	Indicator green	_____
Anaesthetic filter	Check length of use	Replace filter if necessary	_____
Flowmeters	Open flow-control valves, O ₂ first, keep O ₂ open, then N ₂ O	Flow available	_____
	Switch to Air; check flow	Air flow available	_____
	Switch to N ₂ O		_____
Low Flow (ORC)	Close O ₂ flow-control valve,	N ₂ O flow 0.5 to 0.8 L/min	_____
Oxygen Ratio Controller	Open N ₂ O flow-control valve completely		_____
	Set an O ₂ flow to 1,5 L/min	N ₂ O flow 3 to 5 L/min	_____
	Open N ₂ O flow-control valve completely		_____
O₂ flush (bypass)	Press lever or button	Flow occurs	_____
Vapor 19.n	Zero setting	Locked	_____
	Filling level	Enough	_____
	Period since last inspection	Not longer than 6 months (check label on Vapor)	_____
Safety filling system	Sealing block	Pushed in and secured	_____
Filling spout	Valves	Closed	_____
Plug-in system	Connector	Plug-in adaptor is level	_____
	Locking lever	Locked	_____
Selector switch	Switch position	Correct	_____
Interlock	Vapor locking lever	Non-used Vapor blocked	_____
Anaesthetic monitor	Calibration – zero-point*	Completed	_____
	Check of functioning	Function in order	_____
	Anaesthetic agent selection	Correct agent selected	_____
Devapor	Electrical supply	Connected to mains	_____
	Zero setting	Locked	_____
	Filling level	Enough	_____
	Period since last inspection	Not longer than 6 months (see check label)	_____
	Warming-up phase	completed "Operation" light comes on after about 5 minutes	_____
Safety filling system	Filling inlet closed	Points downward	_____
Plug-in system	Connector	Plug-in adaptor is level	_____
	Locking lever	Locked	_____
Interlock	Vapor locking lever	Non-used Vapor blocked	_____
Anaesthetic monitor	Calibration – zero point*	Carried out	_____
	Check of functioning	Functioning in order	_____
	Anaesthetic agent selection	Correct agent selected	_____

* when specified in Instructions for Use

Always check just before use



What	How	Desired	Actual
<p>Circle System 9</p> <p>Soda lime</p> <p>No leaks for semi- closed and semi- open system</p> <p>Pressure-limiting valve</p> <p>O2 analyser</p> <p>Volume monitor</p> <p>Pressure monitor</p> <p>CO2 monitor</p>	<p>Hoses Breathing bag Absorber Volumeter Volumeter heater Airway pressure gauge Measuring connections Valve discs (insp. und exp.) Fresh-gas hose</p> <p>Condition of lime</p> <p>Y-piece, pressure- limiting valve to "MAN" and 70 mbar. Cause pressure to rise to 20 to 25 mbar with O2 flush. Set O2 flow to 0.12 L/min.</p> <p>Pressure-limiting valve to "MAN" and 30 mbar, seal Y-piece. Open O2 flow-control valve completely. Close O2-flow-control valve again.</p> <p>Calibration with Air Check of functioning</p> <p>Calibration – zero-point Check of functioning</p> <p>Calibration – zero-point* Check of functioning</p> <p>Calibration – zero-point* Check of functioning</p>	<p>All present and properly connected</p> <p>Renewed if necessary, no colour change</p> <p>Pressure ≥ 30 mbar for 10 Sekunden</p> <p>Pressure 25 to 35 mbar</p> <p>Carried out, display 21 vol.% O2 In order</p> <p>Carried out In order</p> <p>Carried out In order</p> <p>Carried out In order</p>	<p>If in order tick</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p>
<p>Ventilator</p>	<p>Connections to Circle System 9</p> <p>Switch on, check settings, during inspiration seal Y- piece</p>	<p>All properly connected</p> <p>Ventilation pressure available</p>	<p>_____</p> <p>_____</p>
<p>Suction device</p>	<p>Switch on, seal suction hose</p>	<p>Negative pressure available</p>	<p>_____</p>
<p>Bag for emergency manual ventilation</p>	<p>Check completeness Check of functioning</p>	<p>Complete In order</p>	<p>_____</p> <p>_____</p>

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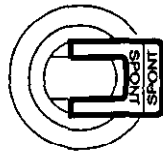
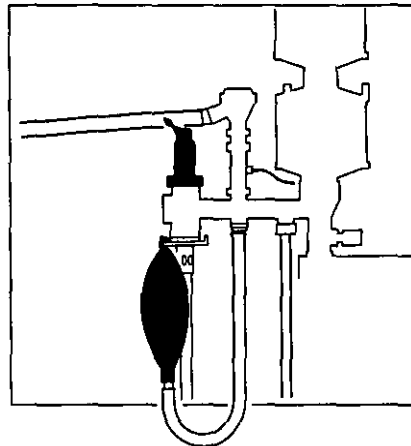
Operating modes

Spontaneous breathing

Sulla 909 V inhalation anaesthetic machine with Circle System 9 re-breathing system – lever on pressure-limiting valve to "SPONT" – breathing bag filled – provides flow of fresh-gas for spontaneous breathing.

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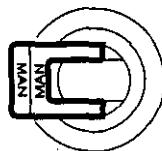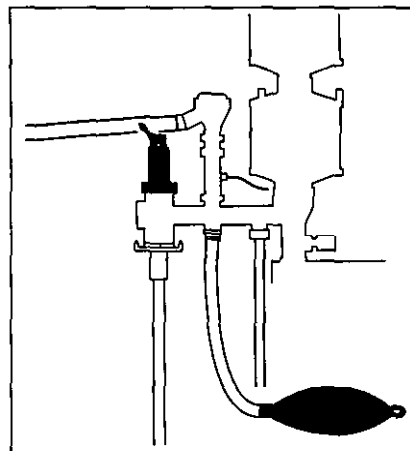
Manual ventilation

Sulla 909 V inhalation anaesthetic machine with Circle System 9 re-breathing system – lever on pressure-limiting valve to "MAN" and pressure limit set on scale – provides flow of fresh gas for manual ventilation – squeeze breathing bag.

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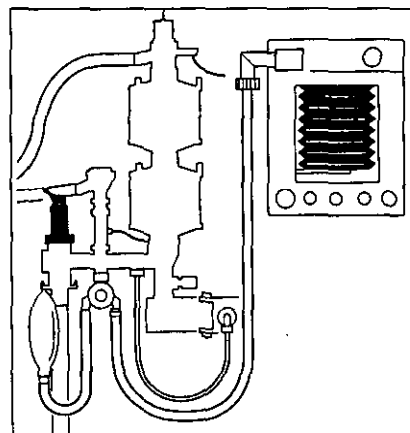
Automatic ventilation

Sulla 909 V inhalation anaesthetic machine with Circle System 9 re-breathing system – lever on pressure-limiting valve to "MAN" and pressure limit set on scale – Ventilog anaesthetic ventilator for automatic ventilation provides flow of fresh gas.

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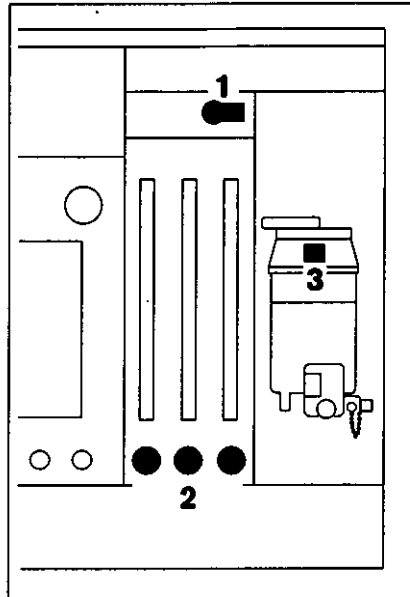
Controlling fresh-gas flow

O₂ + N₂O flow:

- 1 Switch at N₂O position –
- To control O₂ + Air flow, see table page 41:
- 1 Switch at Air position –
- 2 Set ratio of mixture and volume of fresh-gas flow with flow-control valves – then (and not before)
- 3 switch on Vapor –
- connect Y-piece to patient.

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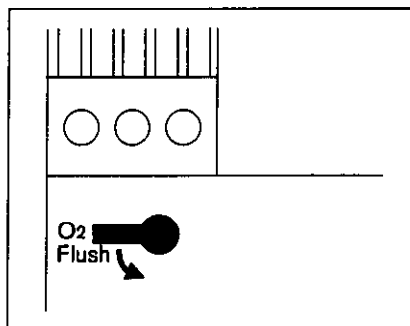
O₂ flush when necessary

- Push lever down: about 55 L/min O₂ flows into circle system to patient – by-passing flowmeter block and Vapor –
- be careful that airway pressure does not rise above permissible limits.

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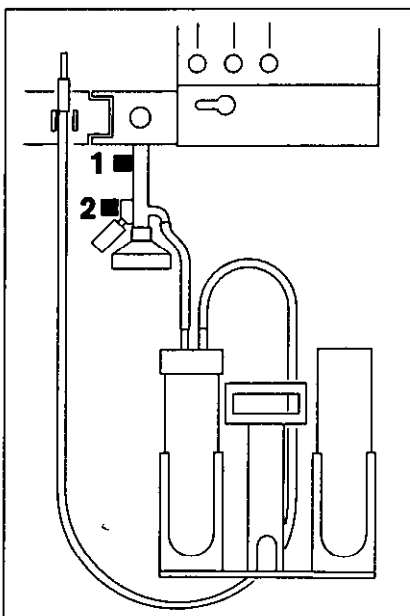Ancillary equipment:

Bronchial suction device

- 1 Open locking valve –
- 2 Set vacuum: keep suction catheter sealed –
- Suck off secretions –
- Dip suction catheter in rinsing liquid – suck rinsing liquid through carefully –
- wenn die Sekretflasche bis 600 mL gefüllt ist:
- 1 close locking valve –
- empty jar –
- observe Instructions for Use for bronchial suction device.

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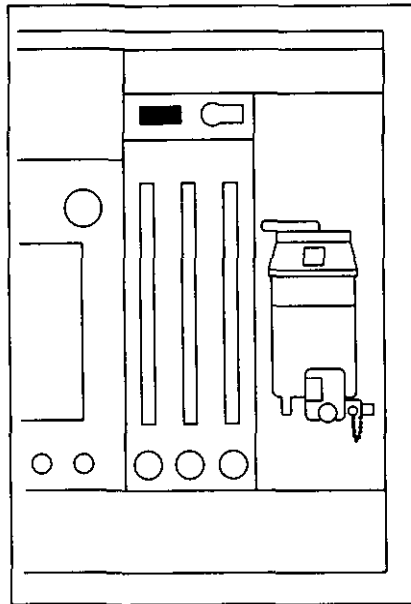
Low Flow anaesthesia

"Low Flow ORC" (ORC = Oxygen Ratio Controller) – safety device is built in – the device is marked with a label.

The Low Flow ORC safety device limits N₂O flow according to O₂ flow, so when the fresh-gas flow is greater than 1 L/min the O₂ concentration in the fresh gas does not fall below 21 vol.%.

At very high flows the N₂O flow is restricted even further.

At very low flows – fresh-gas flows of less than 1 L/min – 0.5 to 0.8 L/min N₂O flows direct to the N₂O flowmeter via a bypass, and can be controlled. Therefore, when the fresh-gas flow is less than 1 L/min manual adjustments have to be made to make sure that the O₂ concentration does not fall below 21 vol.%.

When the fresh-gas flow is in the Low Flow range, the inspiratory O₂ concentration must never fall below 21 vol.%.

Example: At 0.8 L/min N₂O a flow of at least 0.3 L/min must be set manually to make sure that the O₂ concentration in the fresh gas is more than 21 vol.%.

Readjustments must be made if there are fluctuations in the supply pressure.

If the pressure of the O₂ supply falls below 3.0 bar, the O₂ flow may become so low that the O₂ concentration in the fresh gas falls below 21 vol.% – particularly if an O₂ flow of less than 0.5 L/min has been set. However the O₂ shortage alarm is only triggered between 2.0 and 1.8 bar.

O₂ concentration must be monitored continuously with an airway monitor.

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What to do if there is shortage or failure of gas

O₂ shortage

Alarm sounds for at least 7 seconds.

Switch in N₂O position:
N₂O cut-out acts –
machine automatically switches to Air: open Air flow-control valve –

Switch in Air position:
Air flow continues –
N₂O is cut out –
close N₂O flow-control valve –
if O₂ pressure falls below 0.5 bar, N₂O supply to flowmeter block automatically cuts out.
When using machines without an Air supply and no reserve cylinders: ventilate patient with manual ventilation bag.
Vapor handwheel to 0.

Restoring O₂ supply

Push O₂ probe into terminal unit properly
or
open O₂ cylinder – check amount of gas.

Activating O₂ alarm sound

The O₂ alarm sound is activated again,
if at least 2.7 bar has been available
– for at least 20 seconds
– and no O₂ is being supplied.

Air shortage

Float in Air flowmeter drops to lower stop –

switch in Air position:
O₂ flow can be administered again.

Switch in N₂O position:
of no relevance – Air is not administered.

Restoring Air supply

Push Air probe into terminal unit properly.

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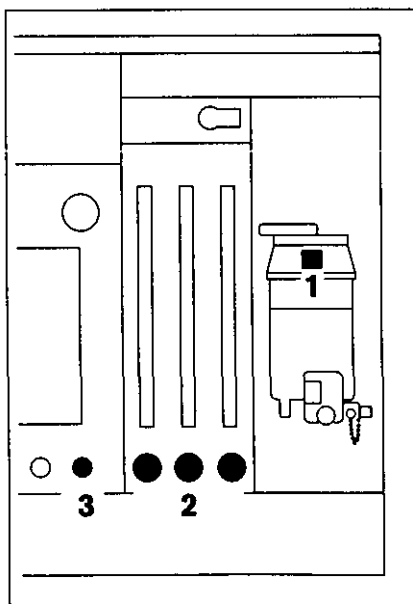
N₂O shortage

Float in N₂O flowmeter drops to lower stop –

Switch in N₂O position:
O₂ flow continues.

Switch in Air position:
of no relevance – N₂O is not administered.

Use alternative or additional anaesthetic procedures.

Restoring N₂O supply

Push N₂O probe into medical gas terminal unit properly.
Open N₂O cylinder – check amount of gas.

Recommended: emergency supply of gas

Dräger O₂ + N₂O cylinder pack, one 2 litre cylinder of each – ask for leaflet 5303.

Shut-down

Follow sequence carefully.

1 Switch off Vapor: handwheel to 0 setting – button engaged –

- Disconnect patient connector –

2 Close flow-control valves – in following sequence:
N₂O – Air – O₂ –

**A continuous flow of fresh-gas through the CO₂ absorber dries out the soda lime and, as a result, CO₂ absorption is reduced.
N₂O causes environmental pollution. Do not use more fresh-gas than is necessary.**

3 Switch Ventillog to 0 –

- O₂, N₂O and Air probes in parking positions or disconnected – Close cylinder valves – Depressurise anaesthetic machine:

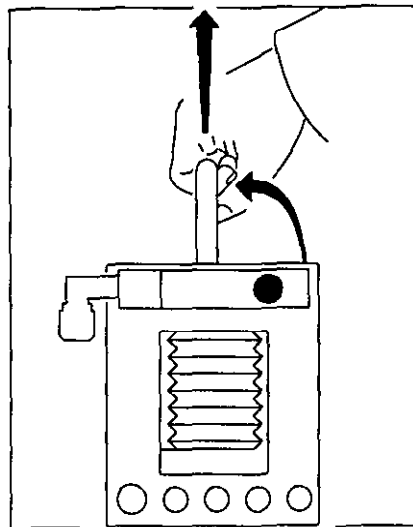
2 Open flow-control valves – in following sequence: N₂O – O₂ – Air.

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Wait until the float in each flowmeter has dropped as far as it can go – close flow-control valves again –

- Switch off airway monitors.
- Care for Circle System, Ventillog, monitors, bronchial suction equipment etc. as described in relevant Instructions for Use.

Stripping down

- Disconnect all probes from the wall terminal units – the hoses can remain with the machine –
- remove cylinder pack from bronchial suction device –
- remove secretion jar with suction hose and flushing jar from mounting and empty –
- take patient system out of Ventillog – pull knob and rotate to the left –
- Vapor should remain on machine.

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Cleaning and disinfecting

- Wipe off obvious dirt with disposable cloth.
- Use surface disinfectants to disinfect – based on aldehydes or quaternary ammonium compounds.

The following should not be used as they may cause damage to materials: substances based on halogen-releasing compounds – strong organic acids – oxygen-releasing compounds.

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Recommendation for users in Germany:

- we recommend the use of disinfectants which are in the DGHM list – this list gives the composition of each disinfectant.
DGHM – German Society for Hygiene and Microbiology – DGHM List – mhp Verlag GmbH, Wiesbaden.
For countries where the DGHM list is not familiar, we recommend the types of disinfectant given above.

Wipe disinfecting

Sulla 909 V basic machine, Vapor 19.n, Ventillog, medical gas connecting hoses and switching valve control line

- do not use alcohol-based disinfectants –
recommended: Buraton 10 F –
Schülke & Mayr, Norderstedt, for instance –
Observe manufacturer's instructions.

Cleaning and disinfecting in machines

Disinfect with moist heat

Connecting hose, bellows, patient system, FGE switching valve, switching valve and waste-gas scavenging connector –

- 93 °C – 10 minutes –
- use cleaning agents only – do not add any disinfectants – parts may be corroded.
- **After disinfecting with moist heat, FGE switching valve must always be sterilized in hot steam at 134 °C.**
Liquid could remain in control channels otherwise and it would not function properly.

Sterilizing

Connecting hose, corrugated bellows, patient system, FGE switching valve and switching valve

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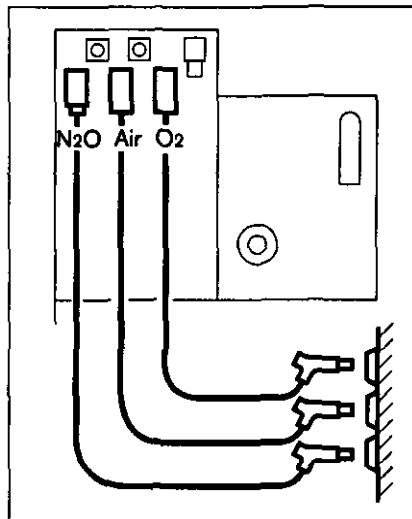
Assembling machine

For Sulla 909 V-D ceiling pendant version, see page 27.

Sulla 909 V: Connecting to medical gas supply

On front of machine

- Close flow-control valves –

On back of machine

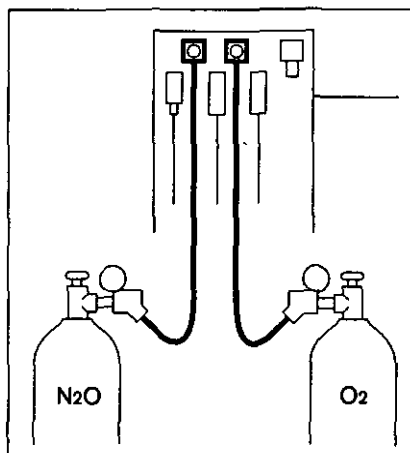
- Screw O₂, N₂O and Air connectors to machine connectors, if no medical air available, leave Air connector free –
- insert probes into terminal units of medical gas pipeline system.

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Using and connecting reserve O₂ and N₂O cylinders

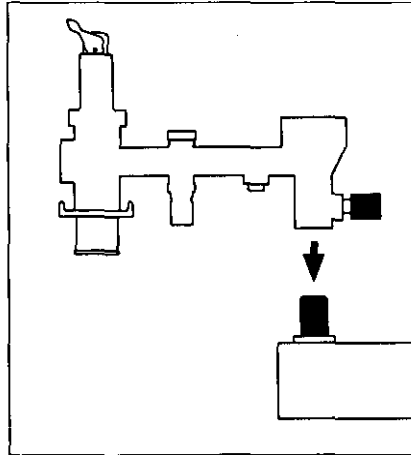
When Sulla is connected to a medical gas pipeline system, cylinders should, nevertheless, be on the machine as a reserve.

At the back of the trolley put cylinders into cylinder holder and secure –

- screw pressure regulators onto cylinder valves –
- connect each pressure regulator to correct gas inlet with gas-specific hose.
- If pipeline supply fails it is then possible to switch immediately to cylinder supply by opening cylinder valves.

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
Connecting Circle System 9

- Get components ready in accordance with Circle System 9 Instructions for Use and assemble –
- fit Circle System and secure with knurled screw –

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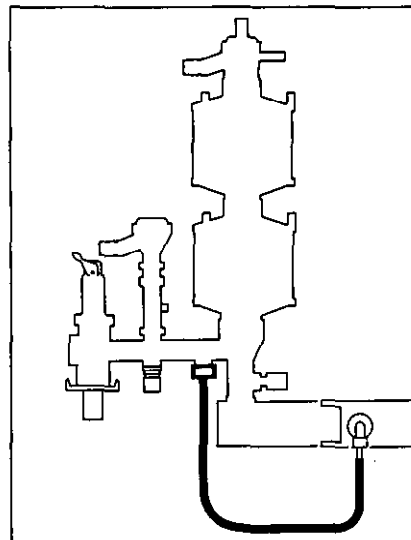if no anaesthetic ventilator is available:

- Connect fresh-gas hose to fresh-gas connector.

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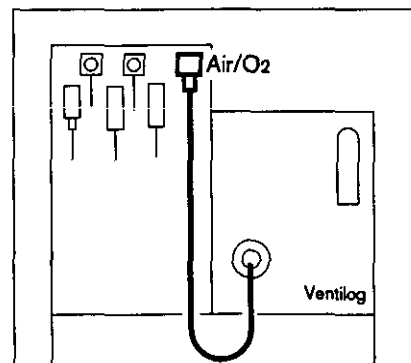
Medical gas supply to Ventilog anaesthetic ventilator

- Connect hose

The Air/O₂ outlet on the inhalation anaesthetic machine supplies Air when the machine is connector to Air from the medical gas pipeline system.

If the pressure of the Air supply falls below 2.0 bar, the outlet will supply O₂. If the pressure of the Air supply then rises to at least 2.7 bar, the outlet will supply Air again.

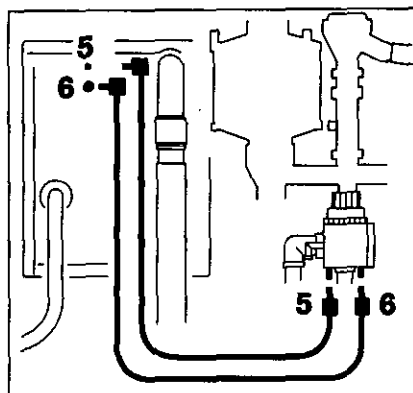
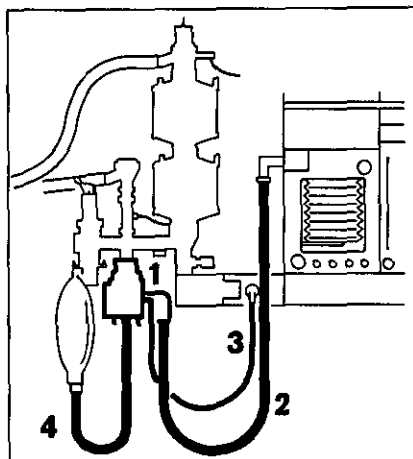
When the machine is not connected to Air from a medical gas pipeline system, the Air/O₂ outlet supplies O₂. Ventilog may then be operated with O₂.

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Ventilation with Ventilog 3

Connecting Ventilog 3 to circle system

Observe Instructions for Use for Ventilog 3.

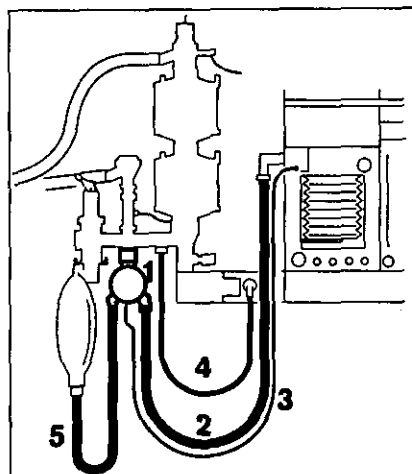
- 1 Screw pneumatic FGE switching valve to circle system arm –
- 2 connect 1m corrugated hose to Ventilog 3 and to FGE switching valve –
- 3 connect fresh-gas hose – from arm of circle system to FGE switching valve –
- 4 connect breathing bag to FGE switching valve with corrugated hose – hang breathing bag on hook –
- 5 attach pneumatic control line – to back of Ventilog 3 and to FGE switching valve –
- 6 attach control line for PEEP – to back of Ventilog 3 and to FGE switching valve – push probes into sockets until they engage.

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Ventilation with Ventilog 2

Connecting Ventilog 2 to circle system

Observe Instructions for Use for Ventilog 2

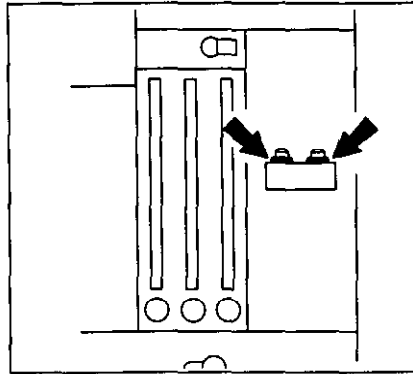
- 1 Screw pneumatic switching valve to circle system arm –
- 2 connect 1m corrugated hose to Ventilog 2 and to switching valve –
- 3 connect pneumatic control line – to nozzle at front of Ventilog 2 and nozzle at back of switching valve –
- 4 connect fresh-gas hose –
- 5 connect breathing bag to switching valve with corrugated hose – hang breathing bag on hook.

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Fitting Vapor 19.n anaesthetic vaporiser

- Do not tilt filled Vapor by more than 45 °C –

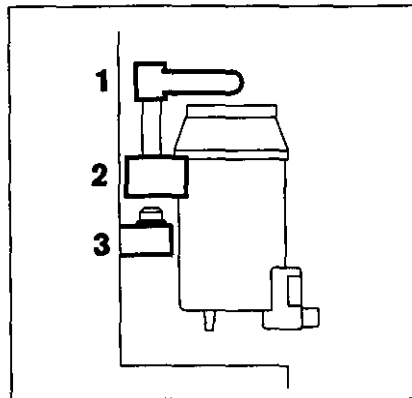
On the base of the Vapor mounting:

- check that there are two O-rings in position and that they are not damaged.

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- 1 Swing locking lever on Vapor to front.

- 2 Position plug-in adaptor on Vapor over two pins and then

- 3 fit to base – must be level.

- 1 Swing locking lever to left – until it engages – Vapor is now locked.

If no Vapor is fitted, the holes in the two pins are sealed, and there is then automatically no connection between the flowmeter block and the fresh-gas outlet.

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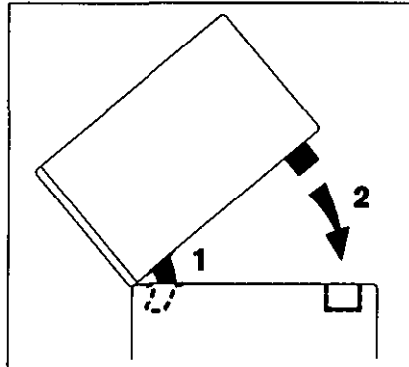
DIN 13 252 requires the following: measurement of expiratory minute volume, airway pressure and inspiratory O₂ concentration – e.g. with PM 8030 or PM 8050 airway monitors.

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
Fitting airway monitor

- 1 Tilt monitor forwards, insert front latches into the slots on the cover plate –
- 2 insert rear latches into the slots and secure at the back with knurled screws.

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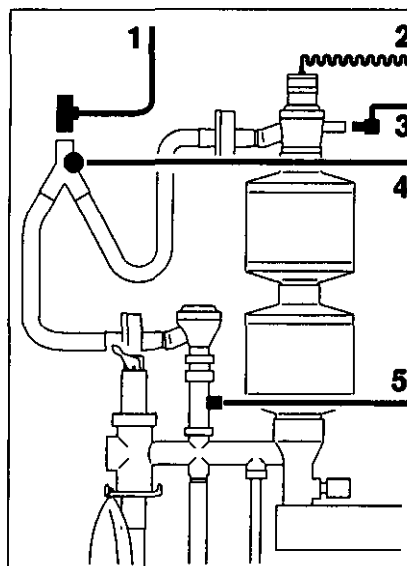Connecting sensors for PM 8050 airway monitor

- 1 T-piece: screw sampling hose firmly to filter and water trap –
- 2 attach O₂ sensor for measuring inspiratory O₂ –
- 3 attach pressure-measuring line for measuring inspiratory airway pressure –
- 4 optional: fit temperature sensor into inspiratory side of the Y-piece –
- 5 flow sensor for measuring expiratory volume – is built in – make electrical connection –

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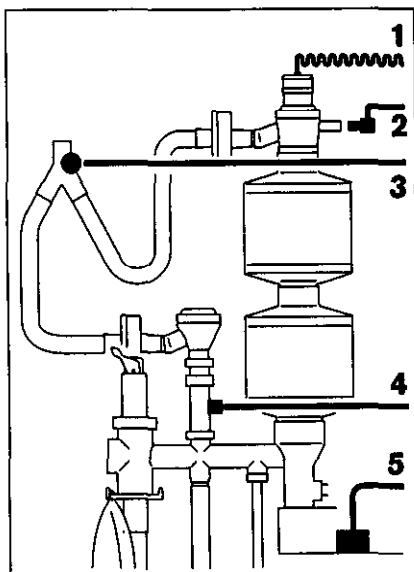
Connect sensor lines to the back of PM 8050 airway monitor in accordance with Instructions for Use for PM 8050

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- Connect mains cable to airway monitor and to power point.

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Connecting sensors for PM 8030 airway monitor

- 1 Attach O₂ sensor for measuring inspiratory O₂ –
- 2 attach pressure-measuring line for measuring inspiratory airway pressure –
- 3 optional: fit temperature sensor into inspiratory side of Y-piece –
- 4 flow sensor for measuring expiratory volume – is built in – make electrical connection –
- 5 sensor for measuring concentration of anaesthetic agent is located in the fresh-gas flow – and built-in on back of arm – make electrical connection –

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Connect sensor lines to back of PM 8030 airway monitor in accordance with Instructions for Use for PM 8030.

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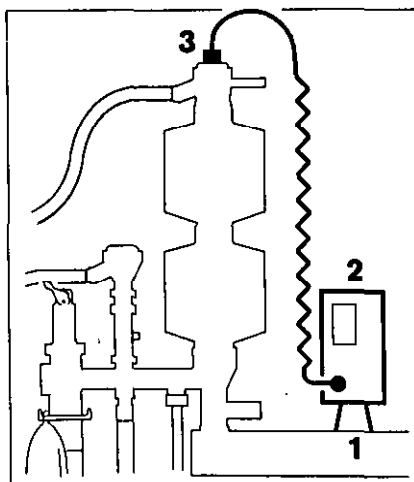
- Connect mains cable – to airway monitor and to power point.

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Fitting Oxydig

If Sulla 909 V is not equipped with ventilator:

- 1 Secure monitor holder to hinged arm –
- 2 fit Oxydig –
- 3 connect O₂ sensor.

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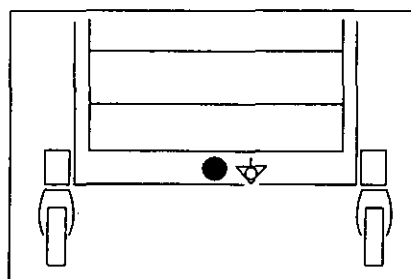
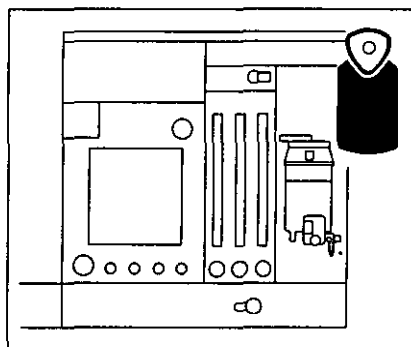
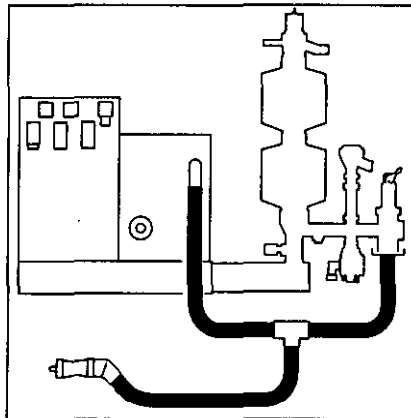
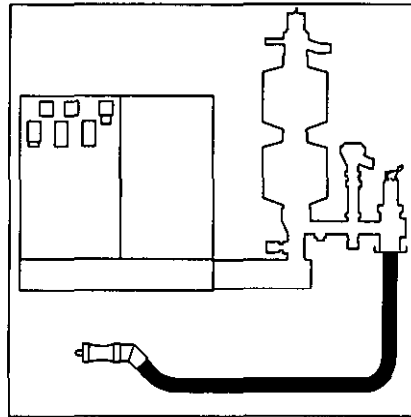
Fitting scavenging system

First illustration – back view, without Ventilog anaesthetic ventilator

Second illustration – back view, with Ventilog anaesthetic ventilator

- Connect to nozzles as shown and connect to T-piece.

If a scavenging system is available:

- Attach probe.
- Push probe into terminal unit of anaesthetic scavenging system just before anaesthesia commences.

If there is no scavenging system:

- Anaesthetic filter equipment 2 = attach corrugated hose fitted with a 633 anaesthetic filter to T-piece or to circle system.
Observe Instructions for Use for Anaesthetic Filter 633

Attaching manual ventilation bag

e.g. Resutator 2000 for emergency ventilation –

- Hang bag from top of rail on right.

Potential equalisation

during intracranial and intracardial operations.

Place both anaesthetic machines on one trolley:

- Provide potential equalisation by connecting 83 01 349 cable from pins at back

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of trolley to potential equalisation point in the room.

- Also connect airway monitor to potential equalisation point.

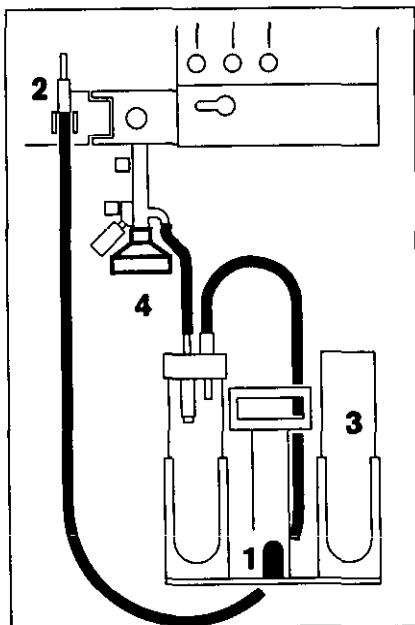
For anaesthetic machines on a ceiling pendant:

- Make sure there is a conductive connection between anaesthetic machine and ceiling pendant and between ceiling pendant and potential equalisation point in room. Equipment installed by Dräger meets these requirements.

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
Preparing bronchial suction device

- 1 Place cylinder pack on the pin –
- Connect hoses –
- 2 fit secretion sight glass –
- 3 fill cylinder with flushing liquid, to flush through the catheter –
- 4 fit bacterial filter.

The drive gas is medical air.
If the Air supply fails, the system switches automatically to O₂.

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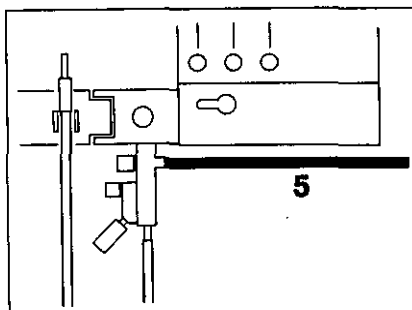Supply from a vacuum pipeline system:

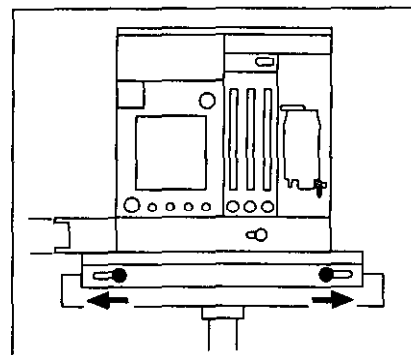
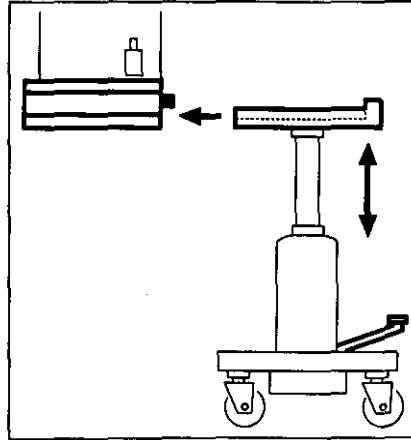
- 5 Fit vacuum hose – connect probe on hose to terminal unit on pipeline system.

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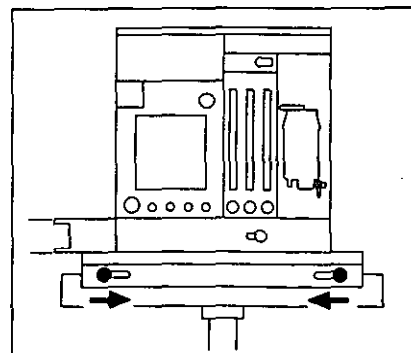
Sulla 909 V-D

- Only use a Dräger lift truck to remove from ceiling pendant, for transport and to re-attach.

Moving Sulla 909 V-D

- Dismantle circle system.
- Disconnect probes on cable and hoses. Unscrew hoses from Sulla 909 V-D.
- Lower ceiling pendant and/or raise lift truck until lift truck platform can be fitted into frame of Sulla 909 V-D easily.
- To raise platform: pump foot pedal with short strokes.
- To lower platform: push foot pedal down as far as it will go.
- Move lift truck platform into the frame of Sulla 909 V-D as far as it will go.
- Slide two locking knobs outwards = unlock Sulla 909 V-D from ceiling pendant, and lock to lift truck.
- Raise lift truck platform a little further and withdraw from the ceiling pendant.
- During transport: lower lift truck platform as far as it will go.

Re-attaching Sulla 909 V-D

- Lower ceiling pendant and/or raise lift truck platform until Sulla 909 V-D can be moved into ceiling pendant mounting.
- Move Sulla 909 V-D into the mounting as far as it will go.
- Slide the two locking knobs inwards = unlock Sulla 909 V-D from lift truck, and lock to the ceiling pendant.
- Withdraw lift truck from mounting.

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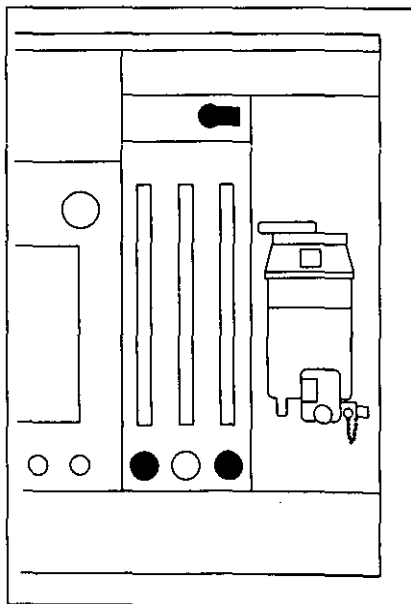
Checking Readiness for Operation

every time when care procedures have been carried out and machine is completely re- assembled and supplied with medical gases – cylinders filled – CO₂ absorber filled.

O₂ cylinders are completely full when the O₂ pressure gauge shows 200 bar. At this pressure, an 11 L cylinder contains 2200 L gas at atmospheric pressure, and a 3 L cylinder contains 600 L.

The N₂O pressure is about 50 bar, as long as the cylinder contains liquid nitrous oxide. While gas is flowing and the temperature drops as a result, cylinder pressure may temporarily fall below 50 bar. It is only possible to determine the amount of nitrous oxide in the cylinders by weighing them. A full 11 L cylinder (full weight 8 kg) contains about 4000 L nitrous oxide at atmospheric pressure and a 3 L cylinder (full weight 2.25 kg) contains about 1125 L.

Checking medical gas connections

All connecting hoses from the medical gas pipeline system and from the cylinders must be tested for a tight and perfect fit and for leaks – visual check.

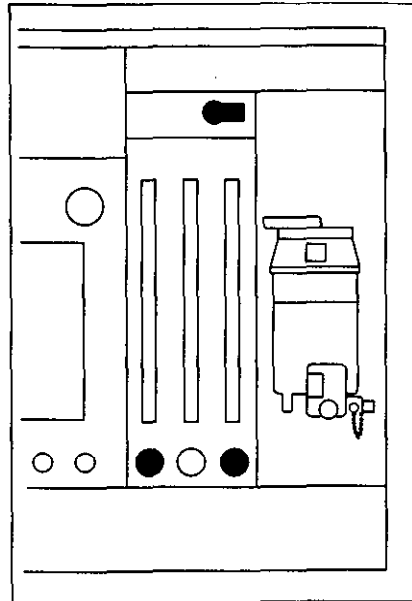
Checking safety devices

ORC, O₂ shortage alarm, N₂O cut-out

- Set switch above the flowmeters to N₂O –
- set O₂ flow to 1.5 L/min –
- open N₂O flow-control valve completely – at maximum setting N₂O flow should be between 3 L/min and 5 L/min – ORC effective –

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
Checking Readiness for Operation

- close O₂ flow-control valve: N₂O flow must fall to 0.5 to 0.8 L/min –
- re-set O₂ flow-control valve to 1.5 L/min, N₂O flow must return to between 3 and 5 L/min –
- check that maximum possible O₂ flow of about 9 L/min can be set –
- interrupt O₂ supply: disconnect probe from medical gas pipeline system or close cylinder valve: after a short time an alarm sound must be triggered which sounds for at least 7 seconds – N₂O flow must fall to zero – N₂O cut-out effective –
- close N₂O and O₂ flow-control valves –
- restore O₂ supply –
- when operating pressure has been 2.7 bar for 20 seconds and no O₂ or N₂O has been used during this time, the O₂ shortage alarm can operate again.

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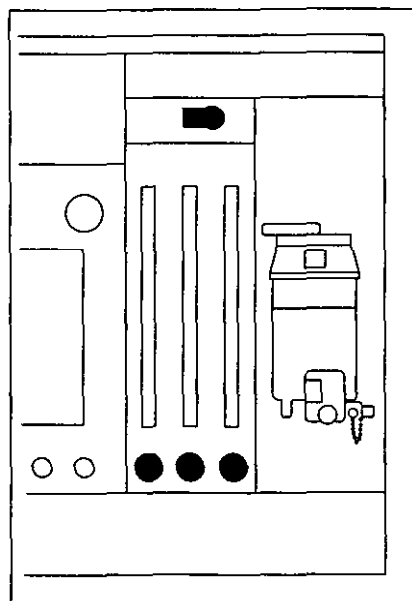
Checking switch – for Air supply

- Switch to N₂O –
- Open O₂, Air and N₂O flow-control valves: O₂ and N₂O must flow – but not Air, which is blocked –
- Interrupt O₂ supply – pull out probe or close cylinder valve: N₂O flow should fall to zero, Air should flow, shortage alarm should sound –
- Restore O₂ supply: N₂O should flow again, Air flow should fall to zero –
- When operating pressure has been 2.7 bar for 20 seconds and no O₂ or N₂O has been used during this time, the O₂ shortage alarm can operate again.

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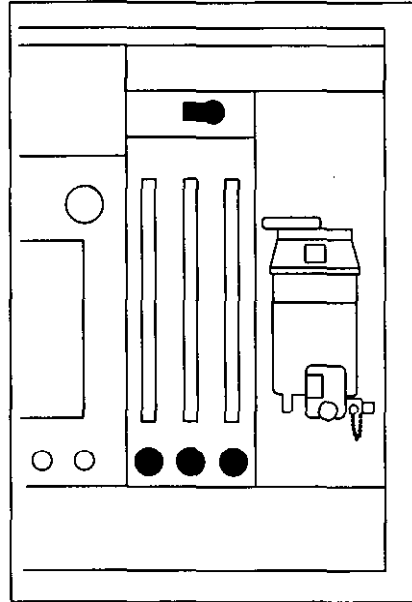
Checking Readiness for Operation

- Switch to Air –
- open O₂, Air and N₂O flow-control valves: O₂ and Air should flow – but not N₂O which is blocked –
- interrupt O₂ supply: O₂ shortage alarm should sound. Air flow should not change, N₂O should be blocked –
- restore O₂ supply –
- When operating pressure has been 2.7 bar for 20 seconds and no O₂ or N₂O has been used during this time, the O₂ shortage alarm can operate again.
- close all flow-control valves.

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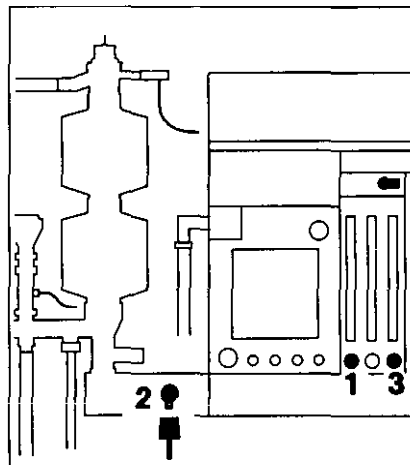Checking gas type

- Switch to N₂O –
- unscrew hose from fresh-gas outlet –
- 1 open O₂ flow-control valve to a flow of 3 L/min –
- 2 Detach O₂ sensor, hold under the fresh-gas outlet and allow gas to flow over it:
- the O₂ concentration shown on the O₂ monitor should obviously be rising towards 100 % O₂ –
- 3 open N₂O flow-control valve to about 3 L/min: the O₂ concentration should obviously fall towards 50 % –
- close O₂ and N₂O flow-control valves.
- re-connect O₂ sensor.

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- Seal patient connector on the Y-piece
- with breathing hose – switch pressure-limiting valve on Circle System 9 to "MAN" and set to 70 mbar –
- build up a pressure of 20 to 25 mbar with the O₂ flush - watch pressure indicator on airway monitor –
- set O₂ flow-control valve to 0.12 L/min:
- pressure – measured on Circle System 9 inspiratory valve – should not be more than 30 mbar.

If not sufficiently leak-tight:

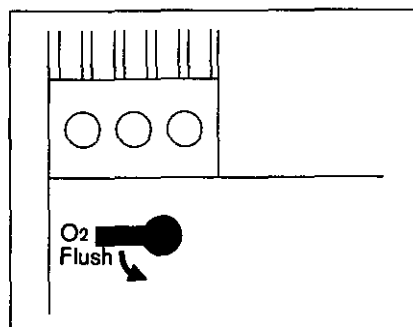
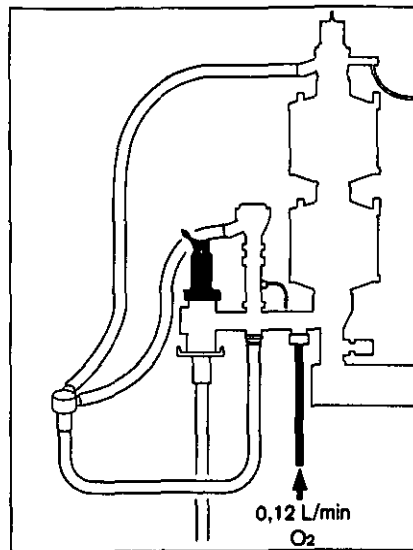
- Remove lime residues from absorber plug-in connectors.
- Check plug-in and screw connectors for tight fit –
- replace any missing or damaged seals. Call DrägerService if necessary.
- Re-open patient connector.

O₂ flush

- Push O₂ flush lever down: Y-piece on circle system should emit strong flow of gas – do not seal Y-piece –
- release lever – automatically returns to position.

Circle System 9

- Check that fresh-gas hose and connections are in perfect condition – visual check –
- Check functioning in accordance with Instructions for Use for Circle System 9.

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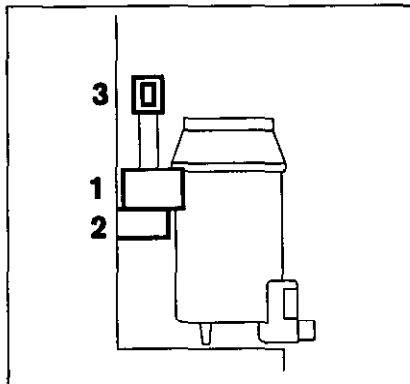
Checking Readiness for Operation

Ventilog

- Ventillog supplied with medical gas? Hose in order?
- Check all connections between Ventillog, pneumatic switching valve, circle system and breathing bag for tight fit –
- Check functioning in accordance with Instructions for Use for Ventillog.

Bronchial suction device

- Check functioning in accordance with Instructions for Use for bronchial suction device.

Vapor 19.n

- Check that:
 - 1 plug-in adaptor is –
 - 2 resting on base evenly and system
 - 3 is locked with lever –
- check filling level – refill –
- check functioning in accordance with Instructions for Use for Vapor 19.n.

Scavenging system

- Check hose connections –
- connect probe to anaesthetic gas scavenging terminal unit – indicator green – system is switched on – or
- has filter been replaced?
- check that filter fits properly into rubber cuffs.

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Checking Readiness for Operation

Manual ventilation bag for emergency ventilation

- Check functioning by pumping manually: when squeezed it should be possible to feel the air flow from the elbow connector and also hear it; when released, the bag should return rapidly to its previous shape –
- Seal elbow connector with the ball of the thumb: it should only be possible to squeeze the bag slightly.

PM 8050 or PM 8030 airway monitors

- Check functioning in accordance with Instructions for Use for the PM 8050 airway monitor or for the PM 8030 airway monitor.

**The machine is now ready for
operation –
continue as on page 12.**

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Clean and disinfect machine and components before each service – and also when returning for repair.

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Inspection and service* 6-monthly by trained service personnel
Regular safety checks 6-monthly by trained service personnel
Pressure regulator basic overhaul every 6 years by DrägerService

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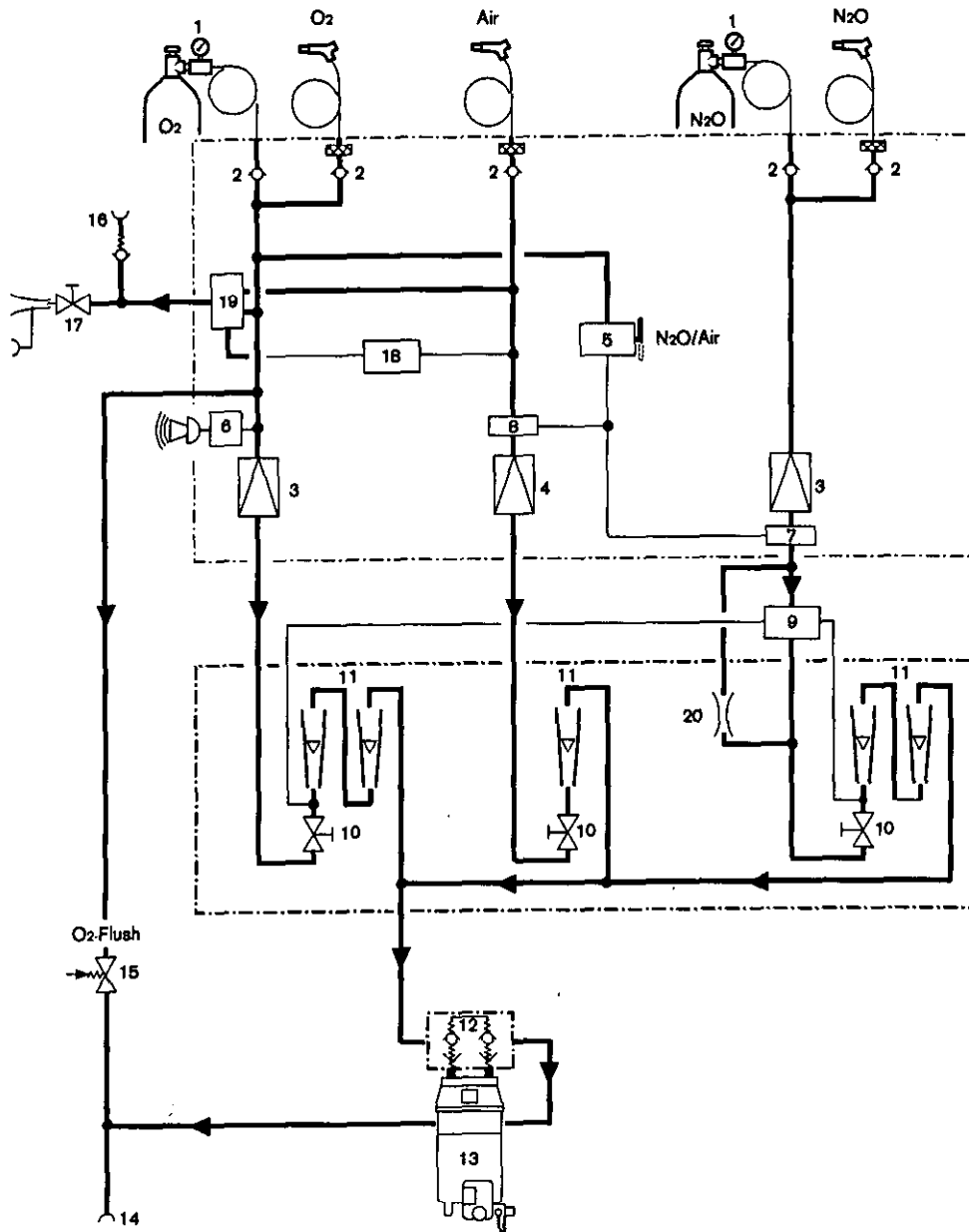
* According to DIN 31 051 the following definitions apply:

Inspection = determining actual condition
Service = measures to maintain required condition
Repair = measures to re-establish required condition
Maintenance = inspection, service and, when necessary, repair

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**Appendix
Diagram of operating principles**

- | | | | |
|---|--------------------------------------|--|---|
| 1 Druckminderer | 11 Flow-Meßröhren | 1 Pressure regulator | 10 Flow-control valves |
| 2 Rückschlagventile | 12 Stecksystem | 2 Check valve | 11 Flowmeters |
| 3 Druckminderer 5/3 bar | 13 Vapor 19.n | 3 Pressure regulator, 5/3 bar | 12 Plug-in system |
| 4 Druckminderer 5/1,5 bar | 14 Frischgasausgang | 4 Pressure regulator, 5/1,5 bar | 13 Vapor 19.n |
| 5 Umschalter "N2O/Air" | 15 O2-Flush | 5 "N2O/Air" switching valve | 14 Fresh-gas outlet |
| 6 Warneinrichtung für O2-Mangel | 16 Steckkupplung für Ventillog | 6 O2 shortage alarm | 15 O2 flush |
| 7 N2O-Sperre | 17 Absaugeejektor | 7 N2O cut-out | 16 Plug-in coupling for Ventillog |
| 8 Druckluft-Ventil | 18 Steuerventil | 8 Medical Air valve | 17 Suction ejector |
| 9 ORC Steuerelement für O2-Mindestkonzentration | 19 Umschalter "Air/O2" (automatisch) | 9 ORC control element for O2 minimum concentration | 18 Control valve |
| 10 Dosierventile | 20 N2O-Bypass | | 19 "Air/O2" switching valve (automatic) |
| | | | 20 N2O by-pass |

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Operating principles

The machine is supplied with the medical gases, O₂, N₂O and Air, from terminal units on a medical gas pipeline system.

As an option, Sulla 909 V may be supplied with O₂ and N₂O from cylinders.

Cylinder pressure is reduced to 5 bar by pressure regulators 1. Check valves 2 prevent the gases from flowing back.

Pressure regulators 3 reduce the O₂ and N₂O pressure to 3 bar, pressure regulator 4 reduces Air pressure from 5 to 1.5 bar.

N₂O and Air can only be administered as alternatives depending on the position of the N₂O/Air switch 5.

The O₂ pressure is monitored by the O₂ shortage alarm 6.

If O₂ pressures fall below 2.0 bar an alarm is triggered, and if they continue to fall, the supply of N₂O is interrupted using the N₂O cut-out 7.

Whatever the position of the "N₂O/Air" switch 5, Sulla automatically switches the medical Air valve 8 to Air.

When O₂ is being used at minimal flow, the Oxygen Ratio Controller, ORC, 9 limits the flow of N₂O so that the O₂ concentration in the fresh gas does not fall below 25 vol.%. When the O₂ flow control is switched off or the supply of O₂ is interrupted, no N₂O is allowed to flow. A flow of Air or N₂O can be set by flow-control valves 10 and a reading taken on flowmeters 11.

The gases are combined upstream of the flowmeters and routed to the anaesthetic vaporiser 13 (Vapor 19.n) via the plug-in system 12 where the anaesthetic agent is added. Even when a Vapor is not connected, gas can flow to the fresh-gas outlet 14.

An O₂ flow of about 55 L/min can be added to the fresh gas using the O₂ flush 15.

The outlet 16 (for driving the ventilator) and the ejector 17 are supplied with medical air if Sulla is connected to medical air. The control valve 18 and the switch 19 bring about an automatic switch to O₂ if Sulla is not supplied with Air or if the pressure of the Air supply falls below 2.0 bar.

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Ambient conditions

During operation	
Temperature	15 to 35 °C (machine should be at ambient temperature)
Atmospheric pressure	700 to 1060 hPa
Relative humidity	20 to 80 %
During storage	
Temperature	-10 to 60 °C
Atmospheric pressure	500 to 1060 hPa
Relative humidity	0 to 80 %

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Performance data

Gas flow

Air/N ₂ O switch for gas type	for use of Air or N ₂ O as alternatives
Air position	O ₂ and Air can be added
N ₂ O position	O ₂ and N ₂ O can be added

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O₂ flow control

fine flow-control valve
5 revolutions
with 2 flowmeters

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Flow range

- | | |
|-------------|--------------------|
| Flowmeter 1 | 0.02 to 0.5 L/min |
| Flowmeter 2 | 0.55 to 10.0 L/min |

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Measuring accuracy

(20 °C, 1013 hPa)

- | | |
|---------------------------------------|---|
| Flowmeter 1 for 0.5 L/min | ±5 % of displayed measured value |
| for < 0.5 to 0.1 L/min range | ±10% of displayed measured value |
| for < 0.1 to 0.02 L/min range | + 20 %, -10 % of displayed measured value |
| Flowmeter 2 for 10 to 0.7 L/min range | ±10 % of displayed measured value |
| for < 0.7 to 0.55 L/min range | + 20 %, -5 % of displayed measured value |

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Technical data

For O₂ flows less than 0.5 L/min

The minimum O₂ concentration of 21 vol.% must be set manually by reducing the N₂O flow.

During N₂O failure

No alarm; float in flowmeter drops to zero.
O₂ may still be administered.

When the anaesthetic machine is also supplied with Air as a third gas,

if one of the gases should fail, the flow of fresh gas is determined by the switch position ("N₂O" or "Air").

Switch in "N₂O" position

O₂ and N₂O may be administered, Air supply is normally blocked automatically.

If O₂ pressure falls below 2.0 bar

Audible alarm for at least 7 seconds.

If O₂ pressure falls below 1 bar

Air supply is automatically released. Air may be administered via the Air flow-control valve.

If O₂ pressure falls below 0.6 bar

N₂O supply is automatically blocked. Lever on switch is not affected and remains in "N₂O" position.

During N₂O failure

O₂ continues to be administered.
No alarm; float in the N₂O flowmeter drops to zero.

Switch in "Air" position

O₂ and Air may be administered, N₂O supply is automatically blocked.

If O₂ pressure falls

Audible alarm for at below 2.0 bar least 7 seconds. Air continues to be administered.

During Air failure

O₂ continues to be administered.
No alarm; float in Air flowmeter drops to zero.

Reactivation of O₂ shortage alarm

after each O₂ failure

When the O₂ supply has been restored, the supply pressure has to be at least 2.7 bar for at least 20 seconds before the O₂ shortage alarm is operational. During this period, do not switch on any uses of O₂, such as O₂ flush, O₂ flow or Ventillog (if driven with O₂).

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Gas supply

From medical gas pipeline system

Pressure at machine connector for O₂, N₂O, Air 2.7 to 6 bar

Screw connectors in accordance with DIN 13 252

O₂ M 12 x 1 female thread
N₂O M 15 x 1 female thread
Air M 20 x 1 male thread

O₂ and N₂O from an additional cylinder supply

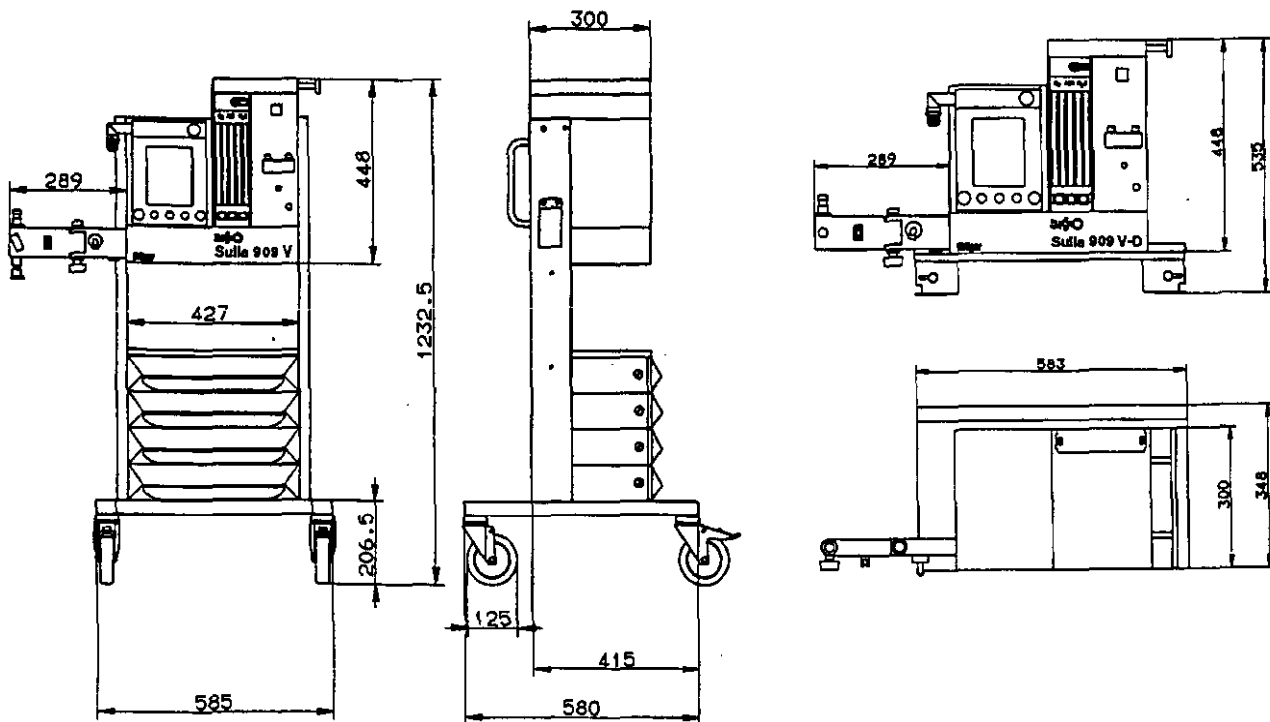
one 11 L cylinder each or one 3 L cylinder each

Pressure at machine connector 5 bar

Weight without cylinders without circle system

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O₂ concentrations which result
for Air and O₂ mixtures

Vol.% O ₂	Flow in L/min (Air + O ₂)													
	1		2		4		6		8		10		12	
	Air	O ₂	Air	O ₂	Air	O ₂	Air	O ₂	Air	O ₂	Air	O ₂	Air	O ₂
21	1.0	0	2.0	0	4.0	0	6.0	0	8.0	0	10.0	0	12.0	0
30	0.9	0.1	1.8	0.2	3.5	0.5	5.3	0.7	7.1	0.9	8.9	1.1	10.6	1.4
40	0.8	0.2	1.5	0.5	3.0	1.0	4.6	1.4	6.1	1.9	7.6	2.4	9.1	2.9
50	0.6	0.4	1.3	0.7	2.5	1.5	3.8	2.2	5.1	2.9	6.3	3.7	7.6	4.4
60	0.5	0.5	1.0	1.0	2.0	2.0	3.0	3.0	4.0	4.0	5.0	5.0	6.0	6.0
70	0.4	0.6	0.8	1.2	1.5	2.5	2.3	3.7	3.0	5.0	3.8	6.2	4.6	7.4
80	0.3	0.7	0.5	1.5	1.0	3.0	1.5	4.5	2.0	6.0	2.5	7.5	3.0	9.0
90	0.1	0.9	0.3	1.7	0.5	3.5	0.8	5.2	1.0	7.0	1.3	8.7		
100	0	1.0	0	2.0	0	4.0	0	6.0	0	8.0	0	10.0		

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To order parts, see Dräger catalogue,
"Parts programme for Anaesthesia, Monitoring and
Ventilation".

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Explanation of abbreviations and terms used

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
Index

Safe connection to other machines.....	4
Safety devices, checking.....	26
Safety procedures.....	5
Scavenging system, fitting.....	23
Service.....	3
Shut-down.....	14
Spontaneous breathing.....	10
Sterilizing.....	16
Stripping down.....	15
Sulla 909 V-D with ceiling pendant.....	25
Technical Data.....	35
Technical safety checks.....	3
Vapor 19.n	
fitting.....	20
testing.....	30
Ventilog	
checking.....	30
supplying with medical gas.....	18
Ventilog 2, connecting to circle system.....	18
Ventilog 3, connecting to circle system.....	19
Wipe disinfecting.....	16

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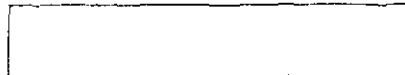These Instructions for Use apply only to
Sulla 909 V/ 909 V-D
with Serial No.:

If no Serial No. has been filled in by
Dräger these Instructions for Use are
provided for general information only and
are not intended for use with any specific
machine or device.

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☐ Moislinger Allee 53 - 55
23542 Lübeck
☎ (04 51) 8 82 - 0
☎ 2 68 07 - 0
FAX (04 51) 8 82-20 80

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© Drägerwerk AG